CE

LS LED

MIMSAL

LS LED

cód. LS010316LED

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LS LED

cod. LS010316LED

Lamp highly versatile. It is suitable for medical examination and has many features that makes it ideal for use in medical centres and veterinary clinics.

The optimum working distance between the light bulb and the object being lit up is 50 cm.

1. Installation

- Requires a MIMSAL mount, which is installed before the lamp is placed in position.
- A table clamp mount is supplied and wall or ceiling mounts or a rotating base can be supplied as optional accessories.
- MIMSAL cannot guarantee the life and functionality of the product where third party accessories are used.

2. Caution

- Always disconnect the lamp from the power supply when cleaning or carrying out maintenance work.
- Clean either dry or using a weak solution of normal soap and water. Avoid using alcohol, acids or any type of organic solvent.
- Do not leave on for longer than 8 hours at a time.

3. Troubleshooting and maintenance

- **Maintenance:** Only for qualified electrical engineers or other qualified personnel.
- **To change the bulb:**
 - Always disconnect the lamp from the power.
 - Replace the bulb.

Please contact your MIMSAL rep if you have any other problem.

MIMSAL only accepts responsibility for maintenance, repair or changes carried out by MIMSAL staff using original MIMSAL parts.

The manufacturer reserves the right to change the characteristics of its products without prior notice.

	LS LED
Light bulb	LED 7,5 W
Head diameter	21 cm
Fitting	E-27 Ceramic
Length of arm	90 cm
Electrical Protection Class	Class II
color	White
Power	220/230V

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