ES

F-803  
Manual de instrucciones  
WE-2018-F803

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## ADVERTENCIA

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# HIGH FREQUENCY & GALVANIC

## WARNING

• Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

• Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.

• Never use the unit outdoors; exposure to the elements can cause damage.

• If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

## EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU.

2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU.

## RECYCLING

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

## WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.





The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

## PACKAGE CONTENTS


• These components are subject to change without prior notice.

• Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY.
1	ELECTRODE 1	10009	1
2	ELECTRODE 2	10010	1
3	ELECTRODE 3	10011	1
4	ELECTRODE 4	10012	1
5	HIGH FREQUENCY HANDLE	10040	1
6	GALVANIC HANDLE	10004	2
7	GALVANIC WIRES		1
8	ACCESSORIES GALVANIC	-	2
9	ACCESSORIES GALVANIC	-	2
10	MAIN UNIT	-	1
11	FEEDER CABLE	-	1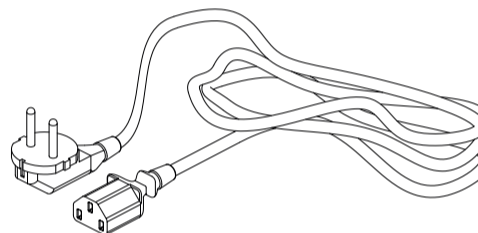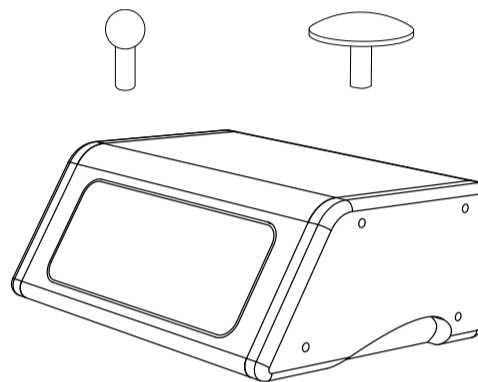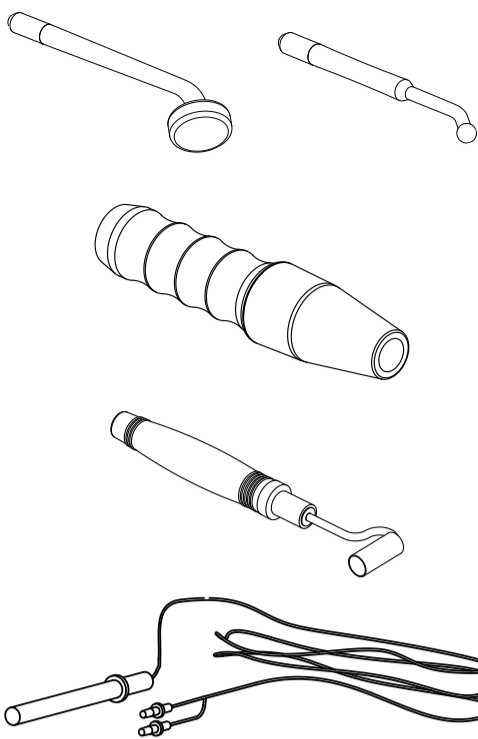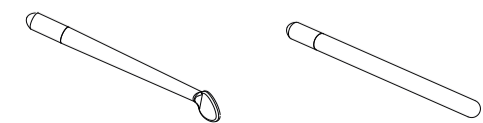

## FRONT PANEL

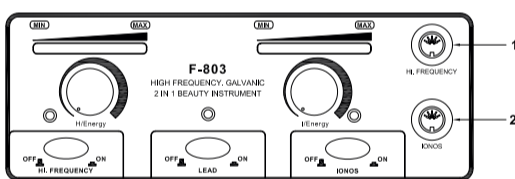1. High frequency handle
2. Galvanic handle

## INSTRUCTIONS OF USE

1. Connect the power supply via the rear panel.
2. Connect the accessories properly to the front/rear panel as indicated in the sketch.

### HIGH FREQUENCY

1. Select the proper glass electrode pipe and insert the metal cap end into the output mouthpiece of the handle (try not to touch the glass electrode with your hands or any metal objects).

Glass electrode pipe assignment position:

a. Mushroom-shaped electrode pipe: used for large areas of skin (face, forehead, neck, etc).

b. Ladle-shaped electrode pipe: used for medium areas of skin (chin, etc).

c. Ball-like electrode pipe: used for small areas of skin (nose, etc).

d. Straight electrode pipe: used for indirect treatment.


2. Switch the unit on by pressing the "H. FREQUENCY" button and make sure the "H/ENERGY" is set to the minimum.

3. Touch the treatment area with the glass electrode. You can then slowly adjust the output using

"H/ENERGY".

4. During the course of the treatment, when changing electrodes you should return the output to the minimum level before swapping the electrodes, and then slowly adjust the energy level and return in to the appropriate intensity required.

5. At the end of the treatment, slowly reduce the "H/ENERGY" level to the minimum output and then switch the "H. FREQUENCY" button off before removing the electrode from the customer's face.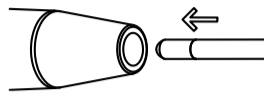

### INDIRECT USAGE

1. Select the straight glass electrode and insert the metal cap end into the output mouthpiece of the handle.

2. Give the handset to the customer, making sure they hold the glass electrode and not the handset.

3. Switch on the unit by pressing the "H. FREQUENCY" button and make sure the "H/ENERGY" is set to the minimum.

4. Slowly adjust the output using "H/ENERGY".

5. Carry out the facial massage or treatment using the prescribed product as required. During the whole treatment process, make sure you do not touch or come into contact with the glass electrode being held by the customer.

6. At the end of the treatment, slowly reduce the "H/ENERGY" level to the minimum output and then switch the "H. FREQUENCY" button off.

## CAUTIONS

• The glass electrode pole will sparkle when it is still less than one inch away from the skin with the instrument switched on. In this case, reduce the energy of the high frequency unit. If the customer still feels the sparkle, they may cover their face with a small, dry cotton towel but never with a nylon or synthetic one.

• Before removing the glass electrode from the skin, decrease the "H/ENERGY" to its minimum setting, then switch the power off before removing it from the skin.

• Customers should not be allowed to wear any metal objects or jewelry; all items must be removed prior to beginning any treatment.


• High-frequency treatments should not be carried out on pregnant women, or any persons using a pace-maker (or any electrical or rhythmic heart-regulating device).

• The products used with the high-frequency treatment must not contain any alcohol or spirit-based substances in order to prevent accidental combustion from arcing.

• Never touch the inner eye (eyeball) area with any high-frequency electrode.

• Only hold the handle of the high-frequency handset.

### GALVANIC

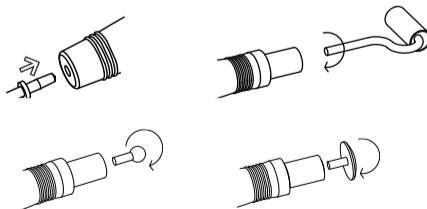## CONTRAINDICATIONS

- Pacemaker (any electrical or rhythmic heart-regulating device)
- Pregnancy
- Thrombosis/Phlebitis
- Skin diseases
- Operations (within the last 6 months)
- Silicone implants
- Hypersensitive or vascular skin

## BEFORE TREATMENT:

1. The treatment area must be washed and dried before any products can penetrate into the tissue.

2. The customer holds the metal pole and the therapist holds the handle of the roller.

3. The polarity of the probes can be changed by adjusting the "LEAD" button between (+R) or (-G).

4. Cover the roller/ball with a piece of cotton or a sponge that has been dipped into a suitable lotion (according to the treatment). The lotion can also be applied directly to the skin.

## STARTING TREATMENT:

1. Turn on "IONOS" and ensure that "I/ENERGY" regulator is set to the minimum output setting.

2. Select the operation by pressing "LEAD".

3. Touch the treated part with the roller/ball.

4. Slowly adjust "I/ENERGY" to the appropriate intensity at which the customer feels a gentle sting. Smoothly slide the roller. The cotton must retain its moisture all the way for the best effect.

Positive pole (anode)	Negative pole (cathode)
+R	-G
Roller/ball (+), metal pole (-)	Roller/ball (-), metal pole (+)
Acid pH solutions	Alkaline pH solutions

## CAUTIONS

• Always make sure customers remove any jewelry, and confirm that they have not had any metal implants (pins, joints, etc.) installed since their last treatment.

• Pregnant women and people with sensitive, injured, inflamed skin or blood capillary expansions should not receive this treatment.

• Customers with health issues such as heart disease or infectious diseases and hyper-allergic or hyper-sensitive skin (see contra-indications) may not use this instrument.

• Always make sure the skin surface has been moistened with plenty of conductive product.

• In the course of using, the roller must not be touch by the metal pole, if any accident happens, the instrument will be protect and turn off the output automatically, at this moment, must turn off the "I/Energy", and restart it from turning on the "I/Energy". When powering off the instrument, please turn off the "I/Energy" first, then the "IONOS".

## TECHNICAL SHEET

Weights and sizes	Packing size (cm)	39x36x18,5
	Gross weight (kg)	4,7
	Product size (cm)	31x17/24x13
Working characteristics	Working power	12 W
	Working frequency (h. freq)	180 KHz
	Working power (galvanic)	5 W
	Working intensity (galvanic)	390 - 1000 micro Amperes
Electrical characteristics	Rated voltage	220V ~ 240V
	Frecuency	50Hz/60Hz
	Rated power	15 W

## FAQ

### GALVANIC

Check the following if the rollers on the probe fail to rotate.

- Dismantle and clean thoroughly.

- If the roller still fails to rotate freely, contact your sales agent.

## MAINTENANCE

### GALVANIC

• Take care not to damage the probe rollers. Never use abrasive substances to clean the probe rollers, since damage to the surface of the roller may scratch the surface of the skin during treatment; any damaged areas may also harbor bacteria and make sterilization more difficult.

• After treatment, all metal roller parts must be cleaned, sterilized and dried. Then store them in the accessory holder to avoid any damage.

### HIGH FREQUENCY

• Glass electrode pipes must be kept clean and dry, and should be sterilized before or after each use.

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